

THESE MATERIALS ARE COMPLIMENTS OF:

Dr. David Violette DDS

34 North Main Street • West Bridgewater, Massachusetts 02333  
508-583-5338 •

# Enhancing your confidence and comfort



## Using a denture adhesive like SUPER POLIGRIP®:

- Holds your dentures securely in place all day long
- Creates a cushion to keep your denture comfortable
- Helps to seal out food particles
- Helps you be more comfortable and confident in social situations
- Super PoliGrip® helps you enjoy more of your favorite foods!®



## Using a denture adhesive is easy.



### Directions for daily use:

1. Clean and dry your dentures
2. Squeeze tube from the top. Start with a small amount of adhesive, apply to dentures in dashes or strips (as shown), not too close to denture edges.
3. Rinse mouth before inserting dentures.
4. Press dentures in place, hold firmly and bite down for a few seconds.



All varieties of Super PoliGrip® feature the Ooze-Control Tip™, to make applying adhesive easier with more control and less ooze.

Remember to see your dentist regularly for a complete oral exam.

**Questions or comments?**  
**Call 1-866-844-2796.**



GlaxoSmithKline  
Consumer Healthcare

**Your Partner in Denture Care**

© 2003 GlaxoSmithKline Consumer Healthcare  
PoliGrip and Polident are registered trademarks of GlaxoSmithKline 3-285ST

THESE MATERIALS ARE COMPLIMENTS OF:

Dr. David Violette DDS

34 North Main Street • West Bridgewater, Massachusetts 02333  
508-583-5338 •

# Keep your dentures clean and fresh with Polident®

## For optimal denture care, follow these steps:

- **Rinse and brush your denture** after each meal to remove loose food particles
  - handle your denture over a sink half-filled with water to cushion it if it drops.
- **Use a specialty denture cleanser** like Polident® every day to:
  - Clean away food particles in cracks and crevices
  - Remove plaque and odor-causing film
  - Eliminate 99.9% of odor-causing bacteria
  - Leave dentures feeling Triple Mint fresh
- **Store your denture** in water overnight to keep it from drying out
  - Add a Polident tablet each night to the water for a convenient way to keep your denture clean, fresh, and odor-free.



## Cleaning your dentures daily is as easy as 1-2-3.

1. Drop one Polident tablet into a denture bath or glass containing enough warm water (not hot) to cover dentures.
2. Allow your dentures or removable bridge to soak in green effervescing solution for 5 minutes (or overnight, for Overnight formula).
3. After soaking in green solution for 5 minutes (or overnight, for Overnight formula), rinse dentures with running water.



**Remember to see your dentist regularly for a complete oral exam.**



*Polident® Cleans Where Brushing Can Miss!®*



GlaxoSmithKline  
Consumer Healthcare

**Your Partner in Denture Care**